

Case

Kerecis® Omega3 Wound

Intact fish skin grafts for tissue regeneration

Granulation over tendon

Patient

- 69-year-old male
- Necrotizing fasciitis on right foot developed after a snake bite

Previous treatment

- Extensive debridement to remove necrotic and infected tissue
- Attempted to get coverage of avascular tissue with NPWT and urinary bladder matrix for three weeks
- Failed to achieve granulation to cover tendons

Treatment

- First Kerecis® Omega3 Wound applied
- Wound had mostly leveled out after 1 week
- Kerecis® Omega3 Wound applied weekly
- Further application of Kerecis continued to build tissue over the exposed tendons
- Tendons were fully covered after 3 applications
- Patient referred to plastic surgery for closure with split thickness graft



What is Kerecis® Omega3 Wound?

Kerecis Omega3 Wound is intact Icelandic fish skin that is identical to human skin and is used for tissue regeneration. Because there is no risk of a viral or prion transfer from Icelandic cod to humans, the fish skin only needs gentle processing for medical use and maintains its natural structure and elements. The fish skin is a natural microbial barrier.

The superior clinical and economical performance of **Kerecis Omega3 Wound** has been demonstrated in two non-industry sponsored, double-blind, randomized clinical trials and in more than 50 other clinical studies.



Indications for use

- Diabetic ulcers
- Chronic vascular ulcers
- Venous ulcers
- Pressure ulcers
- Draining wounds
- Trauma wounds: abrasions, lacerations, second-degree burns, skin tears
- Surgical wounds: donor sites/grafts, post-Mohs surgery, post-laser surgery, podiatric, wound dehiscence
- Partial and full-thickness wounds

Ordering information

orders@kerecis.com

Item no	Description	Size	Qty
50200S16B2D	Kerecis® Omega3 Wound	Disk, 16mm diameter	10 pcs/box
50200S00B2D	Kerecis® Omega3 Wound	1.75 x 1.75 cm	10 pcs/box
50200S01B2D	Kerecis® Omega3 Wound	3 x 3.5 cm	10 pcs/box
50200F01B2D	Kerecis® Omega3 Wound Fenestrated	3 x 3.5 cm	10 pcs/box
50200S02B2D	Kerecis® Omega3 Wound	3 x 7 cm	10 pcs/box
50200F02B2D	Kerecis® Omega3 Wound Fenestrated	3 x 7 cm	10 pcs/box
50200F04B2D	Kerecis® Omega3 Wound Fenestrated	7 x 7 cm	10 pcs/box
50200S03B2D	Kerecis® Omega3 Wound	7 x 10 cm	10 pcs/box

Room temperature storage. 3 year shelf life.

KERECIS

OUR VISION
To become the world leader in tissue regeneration by sustainably harnessing nature's own remedies

KERECIS REIMBURSEMENT HOTLINE
Phone: (844) KERECIS | (844) 537-3247
FAX: (844) 529-3247
Email: reimbursement@kerecis.com

FDA cleared, US and international patents granted and pending.

KERECIS
2200 Clarendon Boulevard
Suite 1400A
Arlington, Virginia 22201
Phone: (703) 287-8752
Email: info@kerecis.com

www.kerecis.com